

## Troubleshooting

Q. The unit doesn't work.

A1. When a button is pressed, does the blue light on the back illuminate? If not,

A2. Replace the batteries.

A3. If the unit does not work with new batteries, please contact us.

## Contact Details

For USA please email [office@experia-usa.com](mailto:office@experia-usa.com)

For UK and rest of the world email [support@experia-innovations.co.uk](mailto:support@experia-innovations.co.uk)

Telephone: (800) 882-4045

**Please note:** email is our support team's preferred communication and this takes precedence over telephone calls.

**Manufactured proudly in the UK by:** Experia

Acorn Phase 3, High Street, Grimethorpe

Barnsley, South Yorkshire, S72 7BD United Kingdom

© Copyright 2010-2013 Experia

All rights reserved. Experia is a registered trademark.



# IRiS Color Selector



Please follow all warnings on the product and within this manual. If upon receipt of this product, the user suspects that the product is defective, and/or requires service or repair due to malfunction or damage to the product, please contact the manufacturer immediately. Please be aware that improper installation and/or use or misuse of this product may result in injury, death, or adverse reactions to its user. This product is intended to be installed and/or use by a trained professional. Failure to comply with these precautions or with specific warnings elsewhere in this manual violates safety standards of design, manufacture, and intended uses of the product.

For optimal results, please read and follow instructions carefully

Please keep for future reference



**WARNING: CHOKING HAZARD**

Small Parts: Not suitable for children under 3 years.



3+ years

experia **USA**  
multisensory for everyone

Congratulations on your new purchase! The IRiS Color Selector has been designed specifically for people with special needs and should, if used in accordance with these guidelines, offer years of switching learning and fun!

The IRiS Color Selector offers opportunities to develop numerous skills including color recognition, cause and effect, eye-hand coordination, gross and fine motor skills, and many more!

The IRiS Color Selector is an IRiS *Talker* and can control any number of IRiS *Listeners*. Put simply, it is operated by pressing any of the switches.

## Features

- IRiS Wirefree Compatible
- 8 Brightly Colored Switches
- 6.5mm Switch Input Sockets
- Rugged Switch Buttons
- 4 x AA Batteries
- Easy to use
- 1 Year Manufacturers Warranty

## Pack contents

- 1 x IRiS Color Selector
- 4 x "AA" Batteries

## Getting Started

1. Insert the four "AA" batteries into the compartment on the back of the color selector following the indications on the battery tray. Push the battery tray in firmly until both sides "click" into place.
2. Pair the Color Selector to the device you wish to control (next section).
3. Press any button to change the color.
4. Plug any of your favorite buttons into the corresponding 1/4" input jacks on the back side of the color selector, as desired. These are located in the same order as the buttons on the top.

## Pairing to IRiS Talkers

This procedure is split into 2 sections, one for those who do not have an IRiS Master Pairer and for those who do.

***Without a IRiS Master Pairer:*** Every IRiS Listener has a button on it called a pairing button. To get, say, an IRiS Bubble Tube to work with the color selector, first turn on the bubble tube. Next, find the pairing button on the bubble tube (near the power input) and press it. You then need to activate the color selector within 5 seconds (by pressing any button). The IRiS Bubble Tube will now change color every time you press a button if correctly paired. If it does not, then simply repeat the above.

***With a IRiS Master Pairer:*** To get, in this example, the Bubble Tube to work with the color selector, first turn on the bubble tube. Next, find the button on the IRiS Master Pairer that corresponds with the Bubble Tube and press it. You then need to activate the color selector within 5 seconds (by pressing any button). The Bubble Tube will now change color every time you press a button if correctly paired, if it does not then repeat the above. For more detail on the Master Pairer, please refer to that item's instruction sheet.

## Installing Batteries: IMPORTANT: BATTERY INFORMATION

- Please retain this information for future reference.
- Batteries should be replaced by an adult.
- Turn the unit upside down to see the battery compartment.
- Apply pressure to the tab at either side.
- Remove the battery compartment drawer.
- Fit 4 x 1.5v AA batteries into the drawer following the marks thereon.
- Push the drawer back into the battery compartment.
- Check to ensure it is in fully and that both drawer tabs are flush with the battery compartment.

## CAUTION:

- Always follow the instructions carefully. Use only batteries specified and be sure to insert them correctly by matching the + and – polarity markings.
- Do not mix old batteries and new batteries.
- Experia recommend the use of alkaline batteries.
- Remove exhausted or dead batteries from the product.
- Remove batteries if product is not to be in use for two weeks or longer.
- Do not short circuit the supply terminals.

Should this product cause, or be affected by, local electrical interference, move it away from other electrical equipment. Reset (switching off and back on again or removing and re-inserting batteries) if necessary.

**RECHARGEABLE BATTERIES:** Do not mix these with any other types of battery. Always remove from the product before recharging. Recharge batteries under adult supervision. **DO NOT RECHARGE OTHER TYPES OF BATTERIES (i.e. Alkaline).**

## Cleaning and Maintenance

- The unit should be cleaned with a damp soft cloth. A non abrasive cleaning agent may be used for stubborn stains. **Ensure no liquid enters the unit.**
- This product has been designed for use by people with special needs, however this does NOT mean it is unbreakable. You must ensure the unit is not hit with anything and that no one drops or hits the unit.