

Cleaning and Maintenance

The fiber optics should be cleaned at least weekly with a damp soft cloth, and your facility's approved cleaning agent, *This may need to be cleaned more frequently depending on the level of usage.*

The Fiber Optic Sideglow should be untangled and straightened during and after every use to ensure it is at its optimum condition and to minimize the chances of someone becoming entangled.

Fiber Optic Safety

A risk assessment must be completed for each person who is to have access to the Fiber Optics before use. Do not allow the Fiber Optics to be put in a user's mouth under any circumstances.

Ensure the person using the Fiber Optics is under constant supervision, and do not allow the Fiber Optics to be put around a person's neck.

Lamp Change

Not required! It uses an LED which lasts for years! Please note: the maximum ambient room temperature in which this unit will operate normally is 27°C, or 80.6°F.

Troubleshooting

Q. The unit doesn't work.

A1. Is it connected to the 110v AC outlet using the correct transformer? If so,

A2. Is the power supply still plugged into the back of the light source? If so,

A3. Is the fiber optic side-glow harness still inserted into the front of the light source? If so,

A4. Please contact us for further assistance.

Contact Details

For USA please email support@experia-usa.com

Telephone: (800) 882-4045


Please note: email is our support team's preferred communication and this takes precedence over telephone calls.

Manufactured proudly in the USA by Experia USA, Inc.

experia **USA**
multisensory for everyone



Circular Fiber Optic Shower - Calming

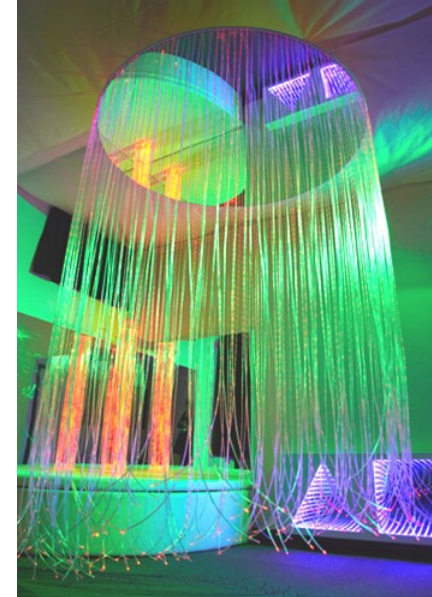

Congratulations on the purchase of your Calming Circular Fiber Optic Shower!

Please follow all warnings on the product and within this manual. If upon receipt of this product the user suspects that the product is defective, and/or requires service or repair due to malfunction or damage to the product, please contact the manufacturer immediately. **Please be aware that improper installation and/or use or misuse of this product may result in injury, death, or adverse reactions to its user. This product is intended to be installed and/or use by a trained professional.** Failure to comply with these precautions or with specific warnings elsewhere in this manual violates safety standards of design, manufacture, and intended uses of the product.

Please ensure you read and follow these Instructions carefully before operating



WARNING: CHOKING HAZARD

Small Parts: Not suitable for children under 3 years.



3+ years


Contents

- 1 x LED Circular Ceiling Shower
- 1 x Low voltage transformer
- 1 x Calming Light Source
- 1 x Fiber optic sideglow harness (installed)
- 1 x Ceiling mounting plate (bolts/washers installed)

Note: Mounting fasteners not supplied, as building codes vary. Consult with your contractor or handyman as to the most appropriate fasteners for your ceiling. **We do NOT recommend that this product be supported by drywall fasteners.** It must be installed into the wood beams above the dry-wall, or onto a separate mounting plate that has been installed into the wooden beams above the dry-wall.

Installation Guide


1. Carefully unpack the Ceiling Shower making sure you have everything listed in the contents section, above. The ceiling mounting plate ships attached to the shower.
2. Remove the ceiling mounting plate by loosening the bolts enough so that you can slide it left or right. Slide until the bolts and washers align with the circular aperture (right), then lift the mounting plate off the Ceiling Shower.

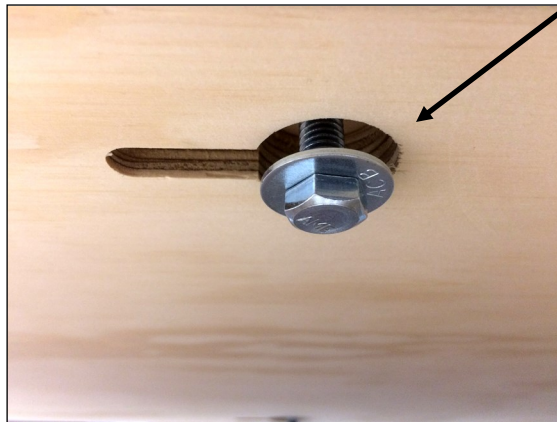

Tools Required

- Ladder (s)
- Adjustable Wrench
- Battery Drill
- Optional: Drill bits

3. Decide where the Ceiling Shower is to be positioned and make sure the ceiling will support the weight of the Circular Shower, as the weight is substantial. If you are unsure, please consult a competent contractor or handyman. Your ceiling may need additional structure added to support this extra weight.
4. Position the ceiling mounting plate where the Ceiling Shower is to be located, preferably aligned with one or more wood studs. Mark the ceiling mounting plate in the appropriate locations to align with your ceiling's studs. Depending on the type of fasteners you are using, you may need to drill pilot holes in the ceiling mounting plate. Make sure enough fasteners are used to support the weight. We recommend a minimum of 4 heavy duty fasteners).
5. Secure the ceiling mounting plate to the ceiling. **NOTE:** Fasteners must be flush with the ceiling mounting plate surface.
7. With the bolts still loose on the ceiling mounting plate (right, below), align the arrow on the top of the circular shower with the arrow on the ceiling mounting plate and lift the Ceiling Shower (this will take two people) so the bolts and their washers align with the circular aperture and drop through the key-hole slots.

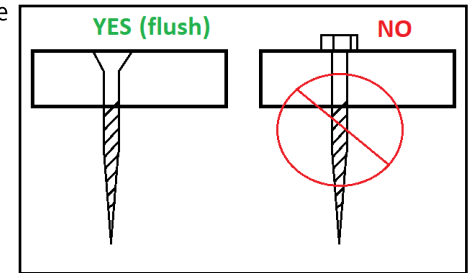

7. Slide the Ceiling shower so that the mounting bolts slide into the slot. The ceiling mounting plate should now be holding the weight of the shower. You can now tighten the bolts.
8. Locate and plug in the power supply into a nearby outlet and into the back of the LED Light Source.
9. Release the fiber optic side-glow strands by carefully cutting the ties holding them.
10. Switch on the electrical outlet and the light source should light up the fiber optic side-glow.



Please note: Only use the transformer supplied. Use of a different transformer may result in damage to the product and void the warranty.